droplet®

MEDLANCE® PLUS Safety Lancets

Safety Lancets

Sterile, single use medical devices intended for capillary blood sampling.

Pressure activated safety lancet. After use, the needle automatically retracts into the housing, protecting user from accidental needlestick injuries.

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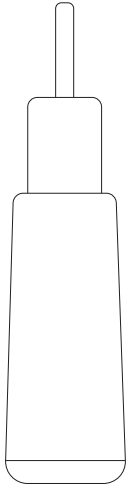
Instructions for use / Instrucciones de uso / Mode d'emploi

⚠ Caution:

For use on a single patient, do not use on more than one patient. Not intended for more than one use. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested. Protective cap is a single sterile barrier system. Do not use the safety lancet if the protective cap has been previously removed from the device. Risk of injury. If the penetration depth is too deep, the puncture can injure children. Before using the lancet for the first time on children, ask your healthcare professional. Start with the lowest penetration depth. Discard the entire used safety lancet after use into a sharps container in accordance with facility guidelines and local regulations.

Please consult with a healthcare professional which size of the device to use for required test blood volume.

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
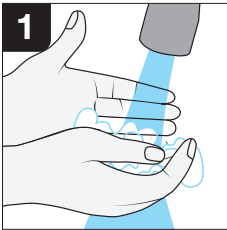
Wash hands with warm water and soap. Dry thoroughly. Disinfect using an alcohol prep pad, ensure the finger is completely dry prior to lancing. Healthcare professionals, wear gloves prior to disinfecting the lancing site and completing the lancing procedure following the facility's protocol.

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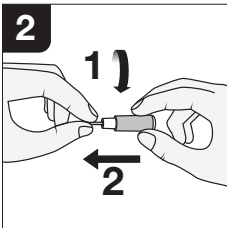
Twist off the protective cap and pull the protective cap straight out.

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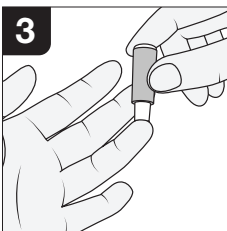
Press the safety lancet body firmly against the puncture site to activate the device. Avoid the finger pads, lancing at the side or top of the finger.

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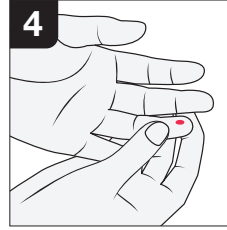Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

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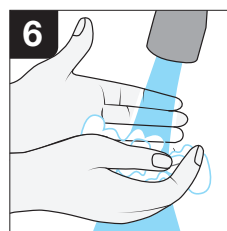
Dispose of the entire safety lancet after use into an approved sharps container in accordance with facility guidelines and local regulations.

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The user should wash hands with warm water and soap after safe disposal of the safety lancet.

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© 2023 HTL-STREFA, Inc., a subsidiary of HTL-STREFA S.A., a company of MTD Group. All rights reserved. Unless otherwise noted the Droplet logo and all other trademarks are the property of HTL-STREFA S.A. and/or its affiliates. A healthcare professional must always rely on his or her own professional clinical judgment when deciding whether to prescribe, recommend, or use a particular product when treating a particular patient.

MFR # 9631
Size 30G x 1.2mm
Activation Contact
Type
Quantities 50 pcs/bx; 8bx/cs

553/05/2023