INSTRUCTION FOR USE

The patients target of this device is the population from infants to elderly. The device can be used both in domestic and professional environment. MR Unsale.

Sterile, single use needles intended for use with pen injector devices for the subcutaneous injection of drugs.

WARNING: Keep out of reach of children without adult supervision. People with reduced mental/physical capacities should be assisted while using the device. Pen needle's size shall be chosen according to your healthcare professional. Here are the steps to follow to ensure accurate use of the pen needle:

- Keep out of reach of children without adult supervision.
- Check the integrity of the seal and of the pen needle. Do not use if damaged. Check the expiry date of the device. Do not use if expired.
- Check the compatibility between the pen needle and the pen injector device to be used for administering the medication, considering the information in this manual and on the packaging.
- Remove the seal from the pen needle. (See Fig 1)
- Connect the pen injector device by pushing the pen needle straight on the pen injector device and twist until it is tight (see Fig. 2a).

WARNING: Pay attention during the connection between the pen needle and the pen injector device. If the needle and the pen injector device are not properly aligned, the drug cartridge needle tip can bend and the medication or dose may not be injected, which may result in serious injury or death.

- Remove the plastic removable outer cover of the blister (See Figures A and 2b). Do not discard, you will need it later to remove the pen needle from the pen injector device.
- Remove the inner protective cap by pulling it straight out, in order to prevent bending of the needle and/or damaging of the needle tip (See Figures B and 3).

WARNING: Remove both the outer cover and the inner needle cover before an injection (See Fig. A and Fig. B, respectively). If both the outer cover and the inner needle cover are not removed before use, the medication or dose may not be injected, which may result in serious injury or death.

- Prime your pen injector device as per your pen's instructions for use, so as to prevent incorrect dosage caused either by air in the cartridge or a clogged needle. If no insulin flow

is seen after several attempts, attach a new pen needle. Always keep a spare pen needle as a replacement (see Fig. 4).

- Set your drug dose (see Fig. 5).
- Make the injection slowly according to the technique and instructions suggested by your healthcare professional performing the injection in a different location each time, in order to avoid lipodystrophy, do not perform the injection in site affected by lipodystrophy. Refer to the instruction on the drugs IFU (e.g drugs temperature). Do not withdraw the needle immediately after the injection, keep it at least for 10 seconds. (see Fig. 6) If, after the needle is removed from the skin, you notice medication dripping, keep the needle inserted longer the next time, at the same dosage. Note: do not massage the injection site after the dose delivery.

Injection techniques:

Recommendation: Injection with a skin fold at a 90° angle. It is recommended that you inject into a skin fold vertically (at a 90° angle) to prevent too deep an injection and hitting the muscle tissue. To make a skin fold when injecting, pinch the skin between the thumb and forefinger and do not pull up the muscle. Injection with a skin fold at a 45° angle. This injection technique is only recommended when you have very little subcutaneous fatty tissue. The minimum angle should be 45°. If the angle is less than 45°, the drug may not reach the subcutaneous fatty tissue and result in swelling or bruising at the injection site. Injection without a skin fold at a 90° angle: if your layer of subcutaneous fatty tissue is very thick, you can make an injection without a skin fold. When you do not use a skin fold, it is recommended that you inject at a 90° angle and choose a short or medium-length needle (up to 6 mm).

AGE	BMI	NEEDLE LENGTH*	
		Without skin fold straight injection	With skin fold angled or straight injection
Pediatric Population (weight greater than 2500g/5,5lb)	-	3.5mm	4mm, 5mm
Slim adults	-	3.5mm	4mm, 5mm, 6mm, 8mm,
Adults	Average	3.5mm, 4mm	5mm, 6mm, 8mm, 12 mm
	Overweight (BMI>25)	4mm, 5mm, 6mm	8mm, 12mm**

* Consult your doctor to choose the most appropriate needle length, method and injection site.

** Also recommended for patients requiring high doses or with limited hand / strength.

Caution

Consult your healthcare professional for assistance in choosing the appropriate needle length and injection technique. Follow the instructions for use, specific storage and handling recommendations of your pen injector device and the patient information leaflet of the drug. Seek immediate medical attention if you have any health concern.

WARNING: Do not change the direction of the pen needle while it remains in the body, as this can result in bending or breaking of the needle.

• At the end of the injection, the outer cover may be used to cover the needle and to avoid accidental punctures during the needle unscrewing and its disposal. (see Fig. 7)

WARNING: Attention should be taken during covering the needles. For needles longer than 6 mm, like G29x12 mm, do not re-cap, in order to avoid that cannula gets stuck in the outer cover.

• Unscrew the pen needle from the pen injector device after each injection, in order to prevent air from entering the cartridge and the insulin from leaking out (see Fig. 8). Finally dispose it in a sharp container respecting applicable regulations and laws.

• This device is single use, single patient only and it is not made with natural rubber latex

WARNING in case of improper uses:

If the needle is left on the pen between injections, it may cause:

- Breakage of the needle.
- Inaccurate dosage since the composition of the medication may be altered.
- Air getting into the cartridge through the needle can reduce the normal flow of medication, causing a reduced drug's delivery.
- The drug's crystallization/chemical degradation, with a partial or complete needle's degradation, causing difficulty in or inability to carry out the injection.
- Contracting infections due to loss of sterility.
- Damage of the needle's tip ("hook" shaped) and loss of needle lubrication, causing an increased pain and possible tearing of the skin.

LOT Lot number

Do not reuse

REF Catalog number

Caution

Use-by date

Sterilized using ethylene oxide
Single sterile barrier system

Cannot be resterilized

Do not use if the packaging is damaged

Consult instructions for use

Recyclable packaging material

Not made with natural rubber latex - natural rubber latex was not used as material in the manufacture of the device

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PICCARE S.p.A.
HTL-STREFA, Inc. - 113 Towne Lake Pkwy, Suite 120 - Woodstock, GA 30188, USA - T: +1-770-528-0410, F: +1-770-528-0411