EN INSTRUCTIONS FOR USE

Droplet pen needles are used with insulin pen injector devices for the subcutaneous injection of insulin in the treatment of diabetes.

N.B. people with reduced mental/physical capacities should be assisted while using the device. Here are the steps to follow to ensure accurate use of the pen needle:

- 1) Check the expiry date of the device. Do not use if expired.
- 2) Check the compatibility between the pen needle and the pen to be used for administering the medication, considering the information on the packaging.
- 3) Remove the seal off the pen needle.
- 4) Attach the pen needle to the pen injector by screwing it on.
- 5) Remove the plastic removable outer cover of the blister (See Fig. A);
- 6) Before starting the injection, remove the plastic removable inner needle cover off the needle (See Fig. B).

WARNING: Remove both the outer cover and the inner needle cover before an injection (See Fig. A and Fig. B, respectively). If both the outer cover and the inner needle cover are not removed before use, the medication or dose may not be injected, which may result in serious injury or death.

- 7) Start the injection and complete it according to the chosen injection technique;
- 8) At the end of the injection, dispose off the needle. Never store the pen with the needle on after use as this would leave an open path of communication between the cartridge and the outside.

The pen needle is a sterile disposable device; therefore, it must be changed after each injection, otherwise you risk:

- Inaccurate dosage. If the needle is left on the pen between injections, the composition of the medication may be altered.
- Air getting into the cartridge through the needle. The air bubbles reduce the normal flow of medication during the injection, with the consequent risk of a reduced dose of the drug.
- The crystallization of the medication. If the needle is left mounted on the pen, the medication remaining inside the pen can transform into crystals, and partially or completely obstruct the needle, causing difficulty in or inability to carry out the injection.
- Contracting infections. Once used, the pen needle is no longer sterile.

• Experiencing a more painful injection. During the injection, the lubricant, put on the needle is completely removed and the tip of the needle undergoes a progressive deformation, losing its original shape. Often the tip of the needle becomes a "hook" shape, causing increased pain and possible tearing of the skin.


- 9) Before each injection, make sure the medication comes out properly. Always keep a spare pen needle as a replacement.
- 10) Use the technique recommended by your doctor for administration.

WARNING: Once the needle is inserted, press the injection button all the way down, being careful not to change the angle of the pen. If, after the needle is removed from the skin, you notice medication dripping, keep the needle inserted longer the next time, at the same dosage.

- 11) After the injection, remove the needle from the pen and dispose of it in an appropriate container respecting applicable regulations and laws.

TO REDUCE PAIN use new, short, small gage needles and always follow your physician's instructions.

Avoid infection. Do not re-use or share pen needles. Do not use pen needles if the individual packaging is damaged, the seal is missing or the needle appears to be damaged. To avoid accidental pricking with needles, never handle someone else's pen needle.

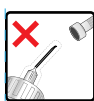WARNING: Remove the inner protective cap by pulling it straight out to prevent bending of the needle and/or damaging the needle tip.

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Agujas medianas

Agujas largas

EN droplet® pen needles are available in 5 versions differing in needle diameter and length to meet your individual needs. You should always consult your physician regarding the most suitable needle length and injection technique to be used (injection angle, with or without pinching).

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EN References obtained from ISO 15223 Medical devices — Symbols to be used with medical device labels, labelling and information to be supplied.

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HTL-STREFA, Inc.